

INSERT WEAR GUIDE

How the best perform

Hu-Friedy

Furcation Straight

Furcation Left

After Five® Left

After Five® Straight

After Five® Right

#3 Beaver tail

Swivel XT™

#1000 Triple Bend

#100 Thin

#10 Universal

Original Prophy Designs

Furcation Right


DIE LINE
WITH ART



*Hu-Friedy's NEW Swivel XT™,
the thin insert that's tough
on heavy calculus*

INSERT WEAR GUIDE INSTRUCTIONS

1. Locate the diagram of the insert to be measured by color.
2. Position point A to point B as shown and/or align the curvature of the tip to match the guide.
3. Rotate insert until tip is flat against the card.
4. Evaluate tip condition, comparing length to red/blue bars.

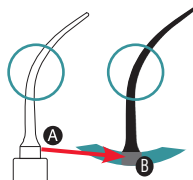

Tip extends beyond blue line
Optimum Efficiency
(Scale Away)



Tip touches blue line
25% Efficiency Loss
(Reorder)



Tip touches red line
50% Efficiency Loss
(Discard)

To reorder inserts call
1-800-Hu-Friedy or your
authorized Hu-Friedy Dealer



©2014 Hu-Friedy Mfg. Co., LLC All Rights Reserved. HF-4661001A

BACK WITH
DIE LINE