

# INSTRUCTIONS FOR USE

## (Sterilization Wrap)

Each wrapper may be used to wrap and sterilize surgical instruments, kits, and/or devices for patient care.

May be used in steam or ethylene oxide sterilization processes.

Wrappers are manufactured for one-time use only

Wrappers are to be inspected for defects prior to use.

Sterilization Wrap should be used in accordance with the preparation, wrapping, and sterilization chamber loading recommendations of the following standards:

ANSI/AAMI ST79: *Comprehensive Guide to Steam Sterilization and Sterility Assurance in Health Care Facilities*


ANSI/AAMI ST41: *Ethylene Oxide Sterilization in Health Care Facilities*

AORN *Standards, Recommended Practices, and Guidelines*

### Common Double Simultaneous Wrapping Technique

Place item(s) on the double-sheet and wrap using typical aseptic wrapping technique.

Ensure that the first fold is pulled far enough to cover all packaging contents. The correct first fold is demonstrated below. **Caution: Covering all package contents with the first fold is required for sterility maintenance, and failure to follow this correct wrapping technique could compromise sterility.**

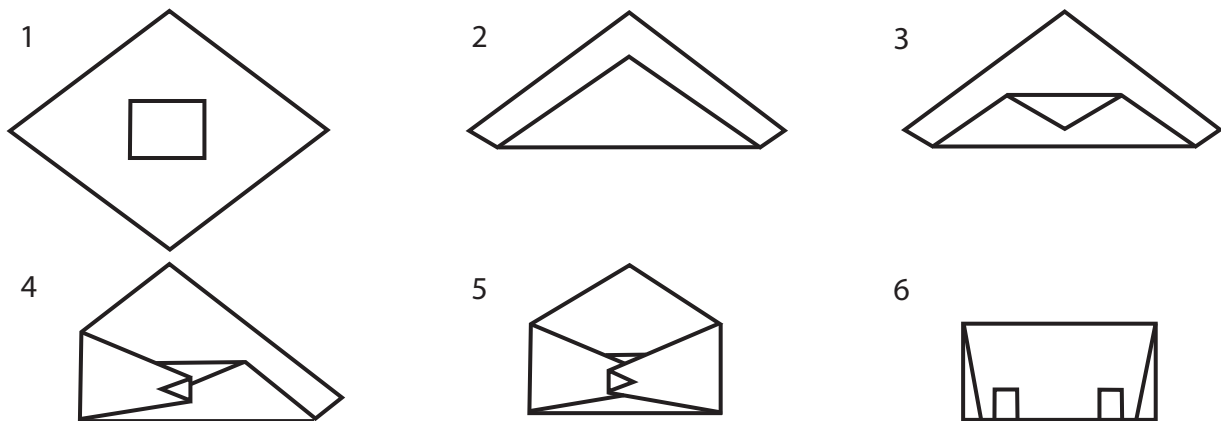

The wrapped package should be secured with the appropriate sterilization indicator tape or with another closure method suitable for the sterilization method to be used. The closure method must allow the sterilant to penetrate the wrapped package, avoid constriction of the packaging, and maintain package integrity.

Follow manufacturer's instructions for use on all sterilizer equipment.

*Please keep these instructions with product for reference.*